VOCO**Grandio[®]SO x-tra****GB**

In accordance with DIN EN ISO 4049

CE 0482**Instructions for use****Product description:**

GrandioSO x-tra is a light-curing, radiopaque nano-hybrid restorative material. It is characterised by an increased depth of cure while at the same time showing reduced shrinkage stress, which means that it can be cured in layers of 4 mm thickness.

GrandioSO x-tra contains 86 % w/w inorganic fillers and is used with a dentine-enamel bond.

Shades:

Universal (U), A1, A2, A3

Indications:

- Class I and II posterior restorations
- Base in class I and II cavities
- Class V restorations
- Locking, splinting of loose teeth
- Repairing veneers, enamel defects and temporary C&B-materials
- Extended fissure sealing
- Restoration of deciduous teeth
- Core build-up

Preparation:

Clean the teeth to be treated. If required mark occlusal contact points. Allow the material to reach room temperature before use.

Shade selection:

Clean the teeth prior to shade selection. The shade is selected in comparison with the tooth while it is still moist.

Cavity preparation:

In principle, cavity preparation should be carried out according to the rules of adhesive filling therapy and should be minimally invasive to conserve healthy tooth substance. Clean and dry the cavity before treatment. Non-carious cervical lesions do not have to be prepared; thorough cleaning is sufficient here.

Creating a dry working field:

Ensure that the work area is sufficiently dry. Use of a rubber dam is recommended.

Matrix placement:

Use of matrices is recommended for cavities with an approximal portion. Translucent matrices that are wedged in the approximal area are advantageous. Minimal separation facilitates the shaping of the approximal contact and insertion of the matrix.

Pulp protection:

A suitable pulp-protective liner that can be covered with a stable cement, if necessary, should be applied in close proximity to the pulp.

Bonding material:

GrandioSO x-tra is used in the adhesive technique with a dentine-enamel bond. Any light-curing bond material can be used. Follow the respective instructions for use with regard to preparation (etching technique) and application.

Application of GrandioSO x-tra:

Apply **GrandioSO x-tra** in layers that are a maximum of 4 mm thick, adapt with a suitable instrument and light-cure afterwards.

Light-curing:

Conventional polymerisation devices with a wavelength in the range of 400 - 500 nm are suitable for light-curing the material. Depending on the light output and shade the curing time for increments of 2 - 4 mm is:

LED-/halogen light	Universal (U)	A1, A2, A3
≥ 1000 mW/cm ²	10 s	20 s
≥ 500 mW/cm ²	20 s	40 s

The curing time for increments of up to 2 mm is 10 s for all shades when using an LED-/halogen light with a minimum light output of 500 mW/cm².

Hold the light emission tip of the device as close as possible to the surface of the filling. Otherwise, the curing depth may be reduced. Incomplete curing may lead to discoloration and discomfort.

Finishing:

The restoration can be finished and polished immediately after removing the matrices (e.g., fine or extra-fine diamond bur, polisher). The tooth should be fluoridated as a final step.

Information, precautionary measures:

- **GrandioSO x-tra** contains methacrylates and BHT. **GrandioSO x-tra** should not be used in case of known hypersensitivities (allergies) to any of these ingredients.
- Phenolic substances, especially preparations containing eugenol or thymol, interfere with the curing of filling composites. Avoid the use of zinc oxide eugenol cements or other eugenol-containing materials in combination with filling composites.
- Each **GrandioSO x-tra** Caps unit should be used for one patient only and should be applied exerting uniform but not excessive pressure.
- Fillings exposed to occlusal forces should be checked at least once a year for early detection of any changes.

Storage:

Store at temperatures of 4 °C to 23 °C. If refrigerated, the material must be allowed to reach room temperature before use. To avoid exposure to light and potentially resulting polymerisation, syringes must be closed immediately after dispensing. Do not use after expiry date.

Our preparations have been developed for use in dentistry. As far as the application of the products delivered by us is concerned, our verbal and/or written information has been given to the best of our knowledge and without obligation. Our information and/or advice do not relieve you from examining the materials delivered by us as to their suitability for the intended purposes of application. As the application of our preparations is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our preparations in accordance with the existing standards and corresponding to the conditions as stipulated in our general terms of sale and delivery.

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VC 60 AA2595 E1 0817 99 © by VOCO
VOCO GmbH