

HOW TO ORDER?



Reference: KITMVX+ADEP

The kit is supplied with a autoclavable cassette.

Find your local dealer online: www.deppeler.ch

📷 [deppeler_instruments](#) | 📘 [Deppeler SA](#)



Dr Monik Vasant

BCHD, MFGDP, MSC

Award winning cosmetic dentist and global lecturer in composite bonding & aesthetic dentistry. Founder of Totally Composite courses, taught to 1900+ dentists across London, Manchester, Dubai, Taiwan and Ibiza.

Finalist, Best dentist, The UK Dental Awards - 2015
50 most influential people in Dentistry, Dentistry Magazine

<https://monikvasant.co.uk/dental-courses/>

📷 [@monikvasant](#)



DEPPELER

Signature of excellence

THE DEPPELER MV AESTHETIC INSTRUMENTS SERIES

The ultimate kit for anterior and posterior composite restorations. Designed by Dr Monik Vasant, Award winning UK cosmetic dentist and global lecturer in composite bonding & aesthetic dentistry.



TTC

TOTALLY COMPOSITE

UNIQUE FEATURES

The 100% Swiss-made instruments exhibit the ultimate in precision, reliance, comfort and surface coating to perform exceptional anterior and posterior composite restorations.

Exceptional sharpness & optimal weight

High tech carbon Shadeline coating

- Prevents from distracting reflections
- Reduces sticking and high resistance to abrasion allowing a more fluid shaping of diverse composite materials

Unique Finest premium steel

 **DEPPELER**



Hand-finish by watchmaking & high-jewellery expert artisans

4 hours+ production time at the Deppeler workshop in Switzerland

Quality check at each of the 15 production steps

Ergonomic design with easy grip soft handles



Pre-Op



Post-Op

HOW TO USE THE INSTRUMENTS



The MV1X – Sculpt & Carry:

This instrument helps sculpt occlusal morphology for posterior restorations as well as carry effects and tints.



The MV2X – Shape & adjust increments in interdental areas:

This sharp interproximal shaper helps access those areas that others can't. Its blade like thickness can be used to shape interdental and awkward areas and handle very small increments easily.



The MV3X – Apply composite & clean away:

This is the instrument for composite veneers. Its large spatula type-end is angled to respect the 3 facial planes of anterior teeth when manipulating material. The other end of the instrument is designed to clean away excess at the proximal and gingival margins prior to curing.



The OP2X – Special large cavities.

This small flat plastic-plugger has been designed to be used when treating larger cavities. Its plugger end is rounded and smooth whilst the flat plastic end is perfectly shaped to place material in small posteriors and anteriors. It can also be used to model the macro texture.



The OP3X – Special small cavities.

This small flat plastic-plugger, of similar shape to OP2X, is aimed for treating smaller cavities.