SOPIRA® Carpule®

Sterile Single-Use Dental Needles

- DE** Gebrauchsanweisung
- GB** Instructions for use
- FR** Mode d'emploi
- ES** Instrucciones de uso
- IT** Istruzioni per l'uso
- PT** Instruções de uso
- NL** Gebruiksaanwijzing
- SE** Bruksanvisning
- DK** Brugervejledning
- NO** Bruksanvisning
- FI** Käyttöohjeet
- GR** Οδηγίες Χρήσης
- LV** Lietošanas instrukcija
- LT** Naudojimo instrukcija


SOPIRA Carpule Needles	Bevel	Colour-Code	Thread	Available as Free-Flow	Available as Blister
0.4 x 38 mm 27 G	long	grey	metric	yes	-
0.4 x 35 mm 27 G	long	grey	metric	-	only blistered
0.4 x 35 mm 27 G	long	grey	inch	-	-
0.4 x 25 mm 27 G	long	grey	metric	yes	yes
0.3 x 25 mm 30 G	long	yellow	metric	yes	yes
0.3 x 21 mm 30 G	long	yellow	metric	-	-
0.3 x 21 mm 30 G	long	yellow	metric	-	only blistered
0.3 x 21 mm 30 G	long	yellow	inch	-	-
0.3 x 16 mm 30 G	long	yellow	metric	-	-
0.3 x 16 mm 30 G	long	yellow	inch	-	-
0.3 x 16 mm 30 G	long	yellow	metric	-	only blistered
0.3 x 16 mm 30 G	short	yellow	metric	yes	yes
0.3 x 12 mm 30 G	short	yellow	metric	yes	yes
0.3 x 12 mm 30 G	short	yellow	inch	-	-
0.4 x 08 mm 27 G	short	grey	metric	-	yes

99000590/02-01.2010

Heraeus

Heraeus Kulzer GmbH
Grüner Weg 11
63450 Hanau (Germany)
info.dent@heraeus.com
www.heraeus-dental.com

CE 0197

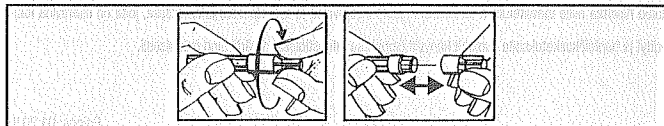SOPIRA® Carpule® Instructions for use **GB**

- 100 sterile, individually packed disposable dental needles for single use
 - Siliconized and sterile.
 - Plastic hub with thread and needle bevel orientation indicator for optimum injection direction and reduced insertion pain.
 - Thin-wall SOPIRA Carpule Free Flow cannula for low injection pressure.

- Use**
- Protect needles against direct sunlight, high temperature and moisture.
 - Only use needles if the seal on the packaging is intact.
 - Needles may be used with all commercial syringes and syringe systems in common use in dental anaesthesia.
 - Before use, unscrew the short, transparent protective cap (removing the long protective cap just before injection).
 - Insert the short needle into the syringe and screw onto the thread of the syringe with the long protective cap on. The needle is properly attached to the syringe when the two parts of the thread (on the syringe and needle) fit together flush (i.e. with no protrusions).
 - Only remove the long protective cap just before injection.
 - Do not bend the needles.
 - Each needle may only be used for one single patient.
 - Avoid all contact with the used needle (risk of infection, do not manually recap).
 - Dispose of needles in special safe containers (puncture-resistant/lockable), which are available commercially.
 - Needles should be collected and protected against human contact (i.e. in a suitable protective container) and then disposed of in accordance with valid regulations.

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