

#niwop



video.wh.com

Now at
your dealer
or wh.com



Happy

patient

Happy

professional



proxeo AURA **Air Polishing System**




It's in your hands

The Proxeo Aura Air Polishing System is a real all-rounder. Soft deposits and stains can be removed thoroughly and efficiently without changing the handpiece. The powders, handpiece and spray heads work together perfectly and allow thorough and atraumatic supragingival and subgingival cleaning.



Supragingival,
sulcus,
subgingival
pockets < 3 mm

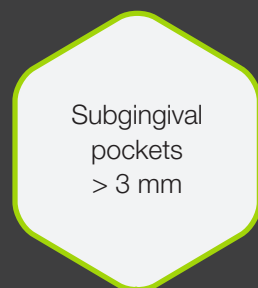


Subgingival
pockets
> 3 mm



Slender – almost like a perio probe

The sterile, slender perio tips allow you to work gently, even in deep pockets. Soft and flexible, they adapt perfectly to the periodontal situation. The lateral outlet directs the jet sideways, promoting atraumatic cleaning.

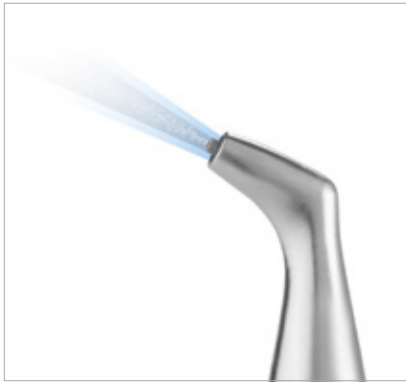

Controlled spray pattern

The optimally bundled powder jet with its focused spray pattern ensures controlled and efficient working. In addition, the powder is enveloped in a water jacket upon exiting, which reduces the powder emission.



Optimal adaptation

The 360° rotation function allows dynamic adaptation of the spray heads to the respective treatment position at any time.



Pretty angled

The special perio spray heads and the 90°- and 120°-angled spray heads in two different lengths ensure you are optimally equipped for all hard-to-reach areas.



The choice is yours

Set the right working pressure to suit the clinical indication using the adjustment ring and achieve brilliant treatment results every time.

- › Prophy: for efficient working in the supragingival region
- › Perio: for atraumatic treatment with glycine powder in the supra- and subgingival region



Convenient refilling

The side opening of the powder container means that instead of holding the handpiece when refilling, you can simply rest it on the table. This leaves both your hands free, making refilling of the powder much easier.



Problem-free workflows

The clear powder container allows you to check the powder level at all times, enabling you to integrate a powder change into your workflow with ease.



4 different turbine connections

The Proxeo Aura air polishing handpiece is available with connections for W&H Roto Quick, KaVo Multiflex, NSK and SIRONA. Simply connect it to the turbine hose and you're ready to start.

Air polishing handpiece PF-60

Air polishing with system

Whether you are working supragingivally, in the sulcus or in deep pockets, the combination of the right powder, the corresponding setting on the adjustment ring and the matching spray head ensures you always achieve optimal treatment results.



Application	Setting	Powder	Spray head
<p>Supragingival</p>	<p>Prophy</p>		<p>Standard Optional</p>
<p>Supragingival, sulcus, subgingival pockets < 3 mm</p>	<p>Perio</p>		<p>Standard Optional</p>
<p>Subgingival pockets > 3 mm</p>	<p>Perio</p>		<p>Standard Optional</p>

Proxeo Aura Air Polishing Powder

Prophylaxis Powder SOFT

Removes supragingival biofilm and stains from the tooth surface gently and effectively.

INDICATIONS

- › Removal of extrinsic stains
- › Cleaning of fissures and cavities
- › Cleaning of orthodontic brackets

Prophylaxis Powder SMOOTH

Uses spherical particles to remove supragingival biofilm atraumatically and protects the tissue at the same time.

INDICATIONS

- › Teeth cleaning with brilliant finish, no damage to tooth enamel
- › Gentle cleaning of restorative materials, brackets, fillings, cavity margins and fissures
- › Treatment of patients on sodium-free diet

Prophylaxis Powder SENSITIVE – Glycine Powder*

Removes supra- and subgingival biofilm atraumatically.

INDICATIONS

- › Gentle removal of stains
- › Biofilm management of periodontal pockets
- › Periodontal maintenance therapy
- › Implant care and treatment of peri-implantitis
- › Cleaning of orthodontic brackets
- › Atraumatic cleaning of restorative materials
- › Minimally invasive on exposed tooth necks and dentine
- › Reduces burden with colonising bacteria



Clinical indications		SOFT	SMOOTH	SENSITIVE
	Removal of supragingival biofilm	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
	Removal of extrinsic stains	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
	Removal of subgingival biofilm			<input type="radio"/>
	Biofilm management of periodontal pockets			<input type="radio"/>
	Care of implant surfaces			<input type="radio"/>
	Treatment of peri-implantitis			<input type="radio"/>
	Cleaning of restorative materials		<input type="radio"/>	<input type="radio"/>
	Care of fillings		<input type="radio"/>	<input type="radio"/>
	Cleaning of brackets	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
	Cleaning of cavity margins prior to etching	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
	Cleaning of fissures prior to sealing	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
	Preparation for bleaching	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>

Clean and hygienic



Cleaning made easy

The special cleaning tools can be used to keep the handpiece and spray heads clean at all times.



Hygienic reprocessing

The handpiece can be simply wipe-disinfected. The removable spray heads are thermo washer disinfectable and sterilizable.

Sets for successful air polishing



Proxeo Aura Propy-Set*

- › Air polishing handpiece
- › 120° spray head
- › Prophylaxis Powder SOFT (250 g)



Proxeo Aura Perio-Set*

- › Air polishing handpiece
- › 120° spray head
- › Spray head perio
- › 40 perio tips (subgingival)
- › Prophylaxis Powder SOFT (250 g)
- › Prophylaxis Powder SENSITIVE – Glycine Powder (160 g)

* only available in EEA and CH



Technical data

Air polishing handpiece	CE 0297	
Operating air pressure bar:		2.2–3.0
Operating water pressure bar:		1.0–3.0
Weight:		155 g
Dimensions (W×D×H):		200×90×50 mm
Ambient conditions:		from +10°C to +35°C relative humidity of 30% to 75%
Type/connection:		PF-60/W&H Roto Quick, PF-60 M/KaVo Multiflex PF-60 N/NSK, PF-60 S/SIRONA
Air pressure P:		800 hPa/1,060 hPa

Spray heads:

Thermo washer disinfectable

Sterilizable up to the stated temperature

Perio tips:

Not for re-use

Sterilized with ethylene oxide



Georg Musil

Product manager W&H, Bürmoos


»Proxeo Aura stands for efficient air polishing. This system with an air polishing handpiece and the corresponding powders allows you to remove supra- and subgingival biofilm, as well as stains – thoroughly and under control.«



Babette McLaren

Dental hygienist, Munich

»The Proxeo Aura is easy to use, ergonomic and suitable for a wide range of applications. Perfect for efficient, supra- and subgingival air polishing!«

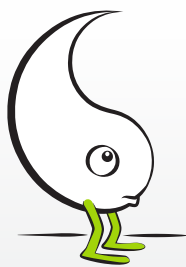

Implantology
without **Periodontology** is
like **Yin** without **Yang**.



#NIWOP: The systematic and evidence-based workflow

W&H Dentalwerk Bürmoos GmbH

Ignaz-Glaser-Straße 53, Postfach 1

5111 Bürmoos, **Austria**

t +43 6274 6236-0

f +43 6274 6236-55

office@wh.com

wh.com