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W8.01-2020

POLO DENT®

Instrument Disinfection CONCENTRATE

en Cleaning and disinfection of instruments
nl Instrumentenreininging en -desinfectie
fr Détergent et désinfectant pour instruments
de Instrumentenreinigung und Desinfection
pl Środek do dezynfekcji i czyszczenia narzędzi
da Desinfektions- og rengøringsmiddel til instrumenter
sv Rengörings- och desinfektionsmedel för instrument
no Desinfeksjons- og rensmiddel for instrumenter
fi Instrumenttien desinfiointi- ja puhdistusaine

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see bottle

Polo MB Oisterwijk BV
Laarakerweg 5
5061 JR Oisterwijk, The Netherlands

Topdental (Products) Ltd
12 Ryefield Way, Sitsden
W. Yorks, BD20 0EF England, UK

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CE

1639

en Cleaning and disinfection of instruments

PoloDent Instrument Disinfection • Disinfecting and cleaning concentrate for instruments, burs and endodontic instruments and items. Chloride, aldehyde and phenol free with a wide material compatibility and broad microbiological efficacy.

Directions for use: Add 20 ml ± 1 ml to 980 ml of clean water for a 2% solution, 40 ml ± 1 ml to 960 ml for a 4% solution. To achieve full efficacy use a 4% solution for a 60 minute soaking time. After disinfection, rinse instruments thoroughly with Reversed Osmosis water or distilled water prior to further processing and sterilisation. Follow the reprocessing instructions from the instrument manufacturer. Change the solution daily. For professional use only.

Efficacy 2%: Bactericidal including MRSA (15 min). Yeastcidal (15 min). All enveloped viruses (15 min) to include: Vaccinia virus, Hepatitis B virus (HBV), Hepatitis C virus (HCV), HIV, Influenza Virus, Coronavirus, Herpesviridae, Filoviridae (Ebola), Rubella, Rabies, Measles.

Efficacy 4%: *As above and to include* Non-enveloped viruses (30 min): Murine norovirus, Adenovirus 5. Mycobactericidal & Tuberculocidal (60 min).

Tested to Standards: EN 14885, EN 14561, EN 14562, EN 14563, EN 14476, EN 14348, EN 13727, EN 13624.

Composition: Aqua, N-(3-Aminopropyl)-N-Dodecylpropane-1,3-Diamine, N,N-Didecyl-N-Methyl-Poly(Oxyethyl) Ammonium Propionate, Alcohols, C9-11, Ethoxylated, Tetrasodium N,N-Bis(Carboxylatomeethyl)-L-Glutamate, Monoethanolamine, Corrosion Inhibitor, Aroma, Dye.

Danger:

H315: Causes skin irritation.
H318: Causes serious eye damage.
H410: Very toxic to aquatic life with long lasting effects.
P280: Wear protective gloves/protective clothing/eye protection/face protection.
P301+P330+P331: IF SWALLOWED: Rinse mouth. Do NOT induce vomiting.

P302+P352: IF ON SKIN: Wash with plenty of water.
P305+P351+P338: IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.

Storage:

P403+P233: Store in a well-ventilated place. Keep container tightly closed.
P235: Keep cool.

P410: Protect from sunlight.

Contraindications:

Conditions to avoid: Heat.

Disposal:

Do not discharge concentrate into drains or rivers.

Any serious incident related to this medical device must be reported to the manufacturer and the competent authorities.

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