i-BASEWAX

EN	Modelling wax
LT	Vaškas modeliavimui
DE	Modellierwachs
BG	Восък за моделиране
CS	Modelovací vosk
DA	Modellervoks
ET	Modelleerimisvaha
FR	Cire de modelage
EL	Κερί μοντελαρίσματος
HU	Modellező viasz
IT	Cera per modellare
LV	Modelēšanas vaskš
NO	Modelleringsvoks
PL	Wosk do modelowania
PT	Cera de modelação
RO	Ceară modelatoare
SK	Modelovací vosk
SL	Modelirni vosek
ES	Cera para modelar
SV	Modelleringsvax
TR	Modelaj mumu
RU	Моделировочный воск
UA	Модельованийний віск

INSTRUCTION FOR USE EN

DESCRIPTION

i-BASEWAX is non-sterile pink thermoplastic plate made of dental wax. i-BASEWAX can be easily softened over a flame, or otherwise, and modelled to the desired shape. Upon cooling, the wax retains its shape with great dimensional stability and excellent adhesiveness to the other dental materials involved.

COMPOSITION

Paraffin 30-100%, pigments <2%.
Product does not contain medicinal substance, including human blood or plasma derivative; tissues or cells, or their derivatives, of human origin; tissues or cells of animal origin, or their derivatives, as referred to in Regulation (EU) No. 722/2012; substances which are carcinogenic, mutagenic, toxic to reproduction or having endocrine-disrupting properties.

PERFORMANCE CHARACTERISTICS

Appearance	Solid
Colour	Pink

INTENDED PURPOSE AND CLINICAL BENEFITS

i-BASEWAX is intended for use during dental treatment procedures.

CLINICAL INDICATIONS

- For the construction of the wax rim (for the record of central occlusion; for the selection of size and position of the anterior teeth);
- For the construction of the wax denture with set artificial teeth ("try in" patients' mouth; to test occlusal contacts).

CONTRA-INDICATIONS

Patients who have a history of severe allergic or irritation reactions to product or to any of the ingredients.

RESTRICTIONS TO COMBINATIONS

None known.

UNDESIRABLE SIDE EFFECTS

In susceptible individuals, product may cause allergic or irritation reactions (skin, eye, mucosa, respiratory tract irritation). Do not use product if patient is allergic to product or to any of the ingredients.

RESIDUAL RISKS

Risk control measures have been implemented and verified, risk is reduced as far as possible, the overall residual risk is judged to be acceptable.

PATIENT TARGET GROUP

No restrictions known regarding patient population, their age and general health conditions. There may be children, middle aged or elderly patient.

INTENDED PART OF THE BODY OR TYPES OF TISSUES OF BODY FLUIDS

Part of the body – mouth. Tissues or body fluids contacted by the device – tooth, saliva, mucosa.

INTENDED USER

i-BASEWAX is developed for professional use in dentistry only. Its user only licensed doctor who has knowledge how to use common dental materials. There is no need for specific training.

STERILITY

i-BASEWAX is delivered non-sterile. There is no need of any preparatory sterilization, cleaning or disinfection, preventive, regular maintenance or calibration to ensure that the device operates properly and safely during its intended lifetime. However, do not use if primary package is damaged.

USE ENVIRONMENT

i-BASEWAX is designed to be used in dental office where ambient temperature is 18-25°C. Do not re-use.

CONSUMABLE COMPONENTS AND ACCESSORIES

No consumable components and accessories are supplied with the device.

INSTRUCTION FOR USE

Construction of the wax rim:

1. Warm the wax plate and make a base on the patient's dental plaster cast model.

2. Warm the other wax plate, twist the roller and press it on the top of the alveolar bone ridge of the cast.
3. Adjust manufactured wax rim by hot wax knife to match the occlusion height.
4. Place the manufactured wax rim into the patient's mouth, ask to clench the wax rollers placed in the central occlusion position.
5. Set the patient's central occlusion height.
6. On clenched wax roller, mark the guidelines for teeth size and position selection: anterior teeth and centre lines, position of canines, the smile and rest lines.

Construction of wax denture with artificial teeth set:

1. Warm the wax plate, apply to the patient's dental plaster cast model.
2. Model wax dental construction. Set artificial teeth upright into wax base, shape artificial gums and any other base.
3. Try the wax dental construction in the patient's mouth, test occlusal contacts.
4. If wax construction fits well, finalize and prepare it for resin polymerization and pressing.

WARNINGS

Do not use product for patients who have a history of severe allergic or irritation reactions to product or to any of the ingredients. Product does not emit radiation and does not cause any electromagnetic interferences.

PRECAUTIONS

Wear protective gloves/protective clothing/eye protection/face protection for doctor and patient.

SHELF-LIFE

Shelf-life of i-BASEWAX is 5 years from the date of manufacture. Do not use after the expiry date. See packing for expiry date. Do not re-use.

STORAGE

Keep product tightly closed in dry well-ventilated place at 5-25 °C. Protect from direct sunlight and heat sources. Do not freeze. Keep out of the reach of children!

DISPOSAL

Dispose of contents/container to as required by national regulatory requirements.

VIGILANCE

If any serious incident that has occurred in relation to the device report to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

SUMMARY OF SAFETY AND CLINICAL PERFORMANCE

i-BASEWAX is safe and performs as intended if it is used in accordance to manufacturer's instruction for use.

MANUFACTURERS RESPONSIBILITY

Our products have been developed for professional use in dentistry. As the application of our products is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our products in accordance with the applied standards.

VALIDITY

Upon publication of this instruction for use, all previous versions are superseded.

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