ONE-STEP®

Universal Dental Adhesive

ONE-STEP® PLUS

Filled Universal Dental Adhesive











ONE-STEP and ONE-STEP PLUS are light-cured, single component adhesives that, unlike others, bond to a multitude of dental substrates, including self-cured and dual-cured materials. ONE-STEP, the clinically proven unfilled adhesive is also available in a filled version, ONE-STEP PLUS. The ONE-STEP and ONE-STEP PLUS patented* formulations utilize some of the same patented chemistry found in ALL-BOND 2°.

UNIQUE BENEFITS

- Bonds to a multitude of substrates
- Compatible with light-, self- and dualcured materials (without the use of additional activators)
- Ensures complete seating of restorations due to low film thickness (Approx. 10µm)
- ONE-STEP PLUS is formulated with filler particles to help reduce post-operative sensitivity

CLINICAL SIGNIFICANCE

- ONE-STEP PLUS unit-dose delivery is easy to use and eliminates potential for cross contamination
- Proprietary chemistry makes it an exceptional adhesive for metal bonding, including implant abutments

UNIVERSAL COMPATIBILITY. NO ACTIVATOR

ONE-STEP and ONE-STEP PLUS can be used for indirect applications or with self- or dual-cured composites and cements and does not require extra steps to achieve compatibility.

	Directs		Indirects	
Products	LC Composite	SC Composite	Cement in LC Mode	Cement in SC Mode
ONE-STEP/ ONE-STEP PLUS	YES	YES	YES	YES
Adper™ Single Bond Plus	YES	NO	Only with Rely X™ Veneer	Only with Rely X™ ARC
OptiBond® Solo Plus™	YES	YES	YES	Only with NX3 resin cement
Prime & Bond® NT™	YES	With DC activator	YES	With DC activator
XP BOND™	YES	With DC activator	YES	With DC activator
ExciTE® F	YES	With DSC activator brush	YES	With DSC activator brush
MPa [™] Direct	YES	YES	NO	NO
Bond-1®	YES	With DC activator	YES	With DC activator

Data compiled from respective manufacturer's instructions for use. Trademarks are property of their respective owners.



BISCO products are proudly made in the U.S.A.

Like us on:







ONE-STEP®

Universal Dental Adhesive

ONE-STEP® PLUS

Filled Universal Dental Adhesive

BONDING TECHNIQUE*







Rinse



Blot and leave visibly moist



Shake bottle and briefly vent, then dispense



Apply
ONE-STEP/
ONE-STEP PLUS



Air dry



Light-cure

PHYSICAL PROPERTIES

ONE-STEP	
Dentin (Ultradent Method)	37(8) MPa
Dentin (#5 gel cap)	26(3) MPa
Enamel (#5 gel cap)	31(3) MPa
ONE-STEP PLUS	
Dentin (Ultradent Method)	37(5) MP
Dentin (#5 gel cap)	24(2) MP
Enamel (#5 gel cap)	27(2) MP
FILM THICKNESS	
ONE-STEP	10+/-2μn
ONE-STEP PLUS	10+/-2μn

Numbers in parentheses indicate standard deviation. Data on file. BISCO, Inc.

ORDERING INFORMATION

KIT CONTENTS

Bico ONE - STE Universal Dental All HEF U-1102 6ff

BOTTLE KITS

ONE-STEP STANDARD PACKAGE WITH SYRINGE ETCHANT

1 Bottle ONE-STEP Adhesive (6ml), 2 Syringes UNI-ETCH® w/BAC (5g ea.),

Accessories, Technique Guide, Instructions/MSDS



ONE-STEP PLUS WITH SYRINGE ETCHANT KIT

1 Bottle ONE-STEP PLUS (6ml), 1 Bottle AQUA-PREP F (2ml),

2 Syringes UNI-ETCH w/BAC (5g ea.), Accessories, Instructions/MSDS



UNIT-DOSE KIT

TOTAL-ETCH/ADHESIVE UNIT-DOSE SYSTEM

80 Pouches each containing 1 ONE-STEP PLUS Unit-Dose with Brush (0.1ml); 100 UNI-ETCH EZ-DOSE™ w/BAC (0.17ml ea.), 1 EZ-DOSE Syringe,

Accessories, Instructions/MSDS



1 Bottle ONE-STEP (6ml), Instructions/MSDS

1 Bottle ONE-STEP PLUS (6ml), Instructions/MSDS

U-1102P U-2001P

U-10010

U-2260K

U-2250K

^{*} Refer to complete instructions for use for technique and instructions.