(fr

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(en)

Composition
Clear, colourless to pale yellow solution of Eucalyptol.
Intended use
Solvent for gutta-percha.
Clinical benefit
Preservation of the tooth functionality.
Indented population
Mature and permanent teeth.
Indication for use

- Fitting of the gutta-percha point in the root canal using cold technique
- Retreatment after root canal filling with gutta-percha.

Contra indication

Hypersensitivity to eucalyptol.

Immature or temporary teeth.

Performance characteristics

Technical performance

Softening and/or dissolving of the gutta-percha.

Clinical performance

Facilitating endodontic procedure in mature teeth.

Instructions for use

Fitting of the point (cold technique)

- Select the gutta-percha point.
- Dip the apical tip into the solvent.
- Insert the tip into the canal and proceed as usual.

Retreatment procedure

- Place two drops of solvent into the pulp chamber.
- Remove the softened gutta-percha.
- Add some fresh solvent from time to time and rinse alternately with sodium hypochlorite.

Warnings and precautions for use

Precautions for use

- Use the liquid within 6 months after first opening.
- Quickly close the bottle after use.
- The device may cause eye irritation and skin burn. Do not handle the device without appropriate personal protective equipment.
- Use a rubber dam to ensure that the solution is not in contact with the mucosa or swallowed by the patient.

Warnings

- Flammable liquid and vapour.
- Do not use a warm technique obturation.

Side effects

Possible hypersensitivity to Eucalyptol.

Disposal

Do not allow the product to enter sewers/ surface water or ground water.

Storage

Store between 5°C and 30°C.

Presentation

1 Bottle of 13 mL of solution.

For dental healthcare professional only

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