Oral & Microsurgery –Soft Tissue & Tongue Control

Handheld Dental Retractors, Dental Cheek Retractors and Tongue Holder/Depressor

CMC MEDICAL DEVICES & DRUGS, S.L
C/ Horacio, Lengo n18, C. P 29006
Málaga-Spain
Phone+34 951 214 054
Email: info@cmcmedicaldevices.com

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en-English

Attention

The operator/user must carefully read and understand this manual thoroughly to keep the product performance durable and reliable for defined life expectancy. After opening the packaging, first of all it is necessary to check the component(s) against the standard configuration. Check that these are all present and in perfect condition.

Intended Use:

These instruments are designed for the controlled retraction, holding, and manipulation of soft tissues, such as the lips, cheeks, and tongue, to enhance visibility and access during dental and oral surgical procedures. They assist in maintaining a clear operative field.

The Instrument Specific Intended Use along with their variants is given below:

Handheld Dental Retractors & Cheek Retractors:

Handheld dental retractors and cheek retractors are intended to be used for retracting and holding soft tissues, including cheeks and lips, to provide a clear field of view during dental, oral and microsurgical procedures. They ensure enhanced access while minimizing patient discomfort.

Variants:

Langenbeck: For retracting soft tissues in oral surgery.

Middeldorp: For precise soft tissue retraction.

Bowdler-Henry's: Provides enhanced access to posterior areas.

Universal Cheek Retractors: Ensures optimal exposure in various dental procedures.

Obwegeser (Internal): For intraoral surgical access.

Farabeuf: For soft tissue retraction in minor oral surgeries.

Minnesota: Provides controlled retraction of the cheek and soft tissues.

Cawood-Minnesota: Combines cheek and tongue retraction functions.

Obwegeser (External): for extraoral surgical applications.

S-Retractor: Offers a versatile design for multiple retraction needs.

Bruenings: for precise surgical tissue retraction.

Sean-Miller: Ensures controlled access in deep surgical sites.

Wassmund: for maxillofacial surgical procedures.

Tongue Holder/Depressor:

Tongue holders and depressors are intended to be used for retracting, holding, or depressing the tongue to improve access and visibility within the oral cavity. They assist in maintaining a clear surgical field and prevent involuntary tongue movement during procedures.

Variants:

Minnesota: for tongue retraction in oral surgery.

Tongue Holder: Provides controlled tongue manipulation.

Cawood-Minnesota: Combines tongue and cheek retraction functions.

 Our Oral & Microsurgery – soft tissue and tongue control instruments are intended to be used by qualified dental/healthcare professionals only in a professional dental/healthcare facility.

Patient population:

These instruments are intended for use on adult and child patients to assist in dental and oral procedures by retracting soft tissues, such as the lips, cheeks, and tongue, to improve visibility and accessibility for the dental professional. The appropriate instrument type and size must be selected based on the patient's oral anatomy and procedural requirements.

For Available Configurations/Models please visit the Retractors section in our Catalogue

Features

Handheld Dental Retractors & Cheek Retractors:

- Engineered with precise angulation to maintain consistent tension on soft tissues, ensuring maximum exposure of the working area without causing excessive strain or discomfort to the patient.

- Fabricated from materials capable of withstanding high-temperature sterilization cycles (up to 134°C) without deformation, ensuring long-term usability and compliance with infection control standards.
- Incorporate a ribbed handle design to prevent slippage during use, even in wet or humid conditions, enhancing procedural accuracy and safety.

Tongue Holder/Depressor:

- Anatomically contoured shape to minimize patient discomfort and ensure effective tongue retraction without causing tissue trauma.
- Features a matte and satin surface to reduce glare under dental operatory lights, enhancing visibility during procedures.
- Smooth, rounded edges to prevent soft tissue damage and ensure safe handling during intraoral use.

Operating instructions

 These instruments are non-sterile and must be cleaned and sterilized before first use.

 Always check if the reprocessed instruments are in perfect condition and do not use the instruments if any kind of deterioration, damage or deformation is observed.

 Use with care to prevent excessive tissue pressure and trauma.

For Handheld Dental Retractors & Cheek Retractors:

- Select the appropriate retractor for the procedure based on tissue type and required access.
- Position the retractor carefully to retract soft tissues without excessive pressure.
- Maintain a stable but gentle hold to prevent slippage and patient discomfort.
- Adjust the angle as needed to optimize access and visibility.

For Tongue Holder/Depressor:

- Choose the appropriate tongue holder or depressor based on patient anatomy and procedural requirements.
- Position the instrument securely to retract the tongue without causing discomfort or trauma.

- Ensure that the tongue remains stable throughout the procedure to maintain a clear surgical field.
- After the procedure, remove the instrument carefully to avoid unnecessary strain on the tissues.

Adverse Event

Any serious incident that has occurred concerning the device should be reported to the manufacturer and to the respective competent authority of the Member State in which the incident has occurred.

Cleaning & Sterilization

Cleaning

Clean the device with care after each application to avoid the drying of contaminants.

Initial Cleaning

1. Remove any visible debris from the instrument using moisten a cotton swab or cloth with isopropyl or ethyl alcohol and gently wiping the surface of the product.
2. Use a soft brush and enzymatic cleaner to remove contaminants thoroughly.

Manual Cleaning

3. Prepare a solution of neutral pH enzymatic detergent according to the manufacturer's instructions.
4. Immerse in a pre-soak enzymatic cleaner solution
5. Thoroughly clean all surfaces of the Instrument using a soft brush or cloth. Pay close attention to any crevices or hard-to-reach areas.
6. Rinse thoroughly under lukewarm running tap water for a minimum of 30 seconds to remove all traces of detergent.

Automated Cleaning

1. Always clean in an appropriate box or cassette and ensure that they are properly positioned within the unit.
2. Always follow the equipment manufacturer's instructions and ensure the instruments are compatible with the cleaning system.
3. Visually inspect for cleanliness, If any contamination is visible, repeat the cleaning steps until required cleanliness is achieved.

4. Dry with clean, lint free cloth or filtered compressed air until there is no visible moisture. Follow STEAM STERILIZATION PROCEDURE.

 Ultrasonic cleaning may be performed in accordance with hospital or facility protocols. Ensure proper rinsing and drying after the process to prevent residue buildup.

Sterilization

Note: Ensure that the instruments are completely dry before sterilization.

Steam Autoclaving

1. Place the instrument in an approved sterilization pouch or wrap.
2. Sterilize using a steam autoclave following a validated sterilization cycle according to ISO 17665-1 and your facility's protocols. A typical recommended cycle is 132-134°C (270-273°F) for 4 minutes.
3. Verify no residual moisture remains after sterilization to prevent corrosion and ensure longevity.
4. Allow the instruments to cool and dry completely before handling. Store in a clean, dry, and controlled environment until ready for use.
5. Follow the pouch/wrap manufacturer instructions for storage conditions and maximum storage time.

Important Notes

- Always Inspect the instruments for any signs of deterioration, breakage and bent etc. after each sterilization cycle. Discard if damage is observed.
- Always follow your facility's infection control protocols.
- Refer to your autoclave manufacturer's instructions for proper operation and maintenance. The Sterilization equipment must be validated by the hospital and or sterilization equipment manufacturers.
- The above-mentioned cleaning and sterilization guidelines, provided by manufacturer are intended as procedures compatible with specific materials. Sterilization must be performed according to the Hospital/Clinic approved protocol.
- The responsibility for ensuring proper sterilization of instruments lies with the user. The effectiveness of sterilization depends on validated cleaning, packaging, and sterilization procedures carried out at the facility.

Disposal

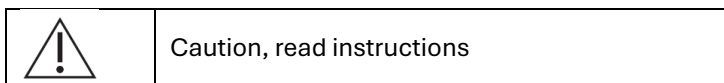
Our Handheld Dental Retractors, Dental Cheek Retractors and Tongue Holder/Depressor contain no hazardous materials. However, these must be cleaned/sterilized as per defined procedures before disposition. Please follow the local and national regulations or healthcare facilities' defined disposal and waste management policies.

Disclaimer

The products must be used, reprocessed, and maintained strictly in accordance with the instructions provided above. Any deviation from these guidelines by the dental professional or user is undertaken at the user's sole risk. The Manufacturer will not accept any requests for refunds or exchanges under warranty for products that have not been handled and reprocessed in compliance with these instructions.

Explanation of utilized symbols

	Medical Device Compliant with Regulation (EU) 2017/745
	Consult instructions for use
	Medical Device
	Product Code/Catalogue Number
	Model/Variant
	Lot Number
	The Device is supplied non-sterile.
	Sterilizable in a steam sterilizer (autoclave) at 132-134°C (270-273°F)
	Manufacturing Date
	Manufacturer
	Authorized representative in the European Community
	Importer
	Keep in a cool, dry place

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