## da Desinfektionsmiddel til roterende instrumenter

**PoloDent Bur Disinfection** • Desinfektionsmiddel- og rengøringskoncentrat til roterende instrumenter såsom bår, diamanter og polerere. Klorid, aldehyd- og phenol-ri.

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1639

## en Rotary instrument disinfectant

**PoloDent Bur Disinfection** • Disinfecting and cleaning concentrate for instruments, burs and endodontic instruments and items. Chloride, aldehyde and phenol free.

**Directions for use:** Add 20ml ± 1ml to 480ml of clean water for a 4% solution. Immerse items in the solution ensuring that all surfaces are covered. Leave for a minimum of 15 minutes (to achieve full efficacy we recommend leaving for 60 minutes). After disinfection, rinse thoroughly with RO or distilled water prior to further processing and sterilisation. Follow the reprocessing instructions from the manufacturer. Change the solution daily. For professional use only.

**Efficacy 4%:** Bactericidal including MRSA (15 min). Yeastocidal (15 min). All enveloped viruses (15 min) to include: Vaccinia virus, Hepatitis B virus (HBV), Hepatitis C virus (HCV), HIV, Influenza virus, Coronavirus, Herpesviridae, Filoviridae (Ebola), Rubella, Rabies, Measles. Non-enveloped viruses (30 min): Murine norovirus, Adenovirus 5. Mycobactericidal & Tuberculocidal (60 min).

**Tested to Standards:** EN 14885, EN 14561, EN 14562, EN 14563, EN 14476, EN 14348, EN 13727, EN 13624

**Composition:** Aqua, N-(3-Aminopropyl)-N-Dodecylpropane-1,3-Diamine, N,N-Didecyl-NMethyl-Poly(Oxyethyl)Ammonium Propionate, Alcohols, C9-11, Ethoxylated, Tetrasodium N,N-bis(Carboxylatomeethyl)-L-Glutamate, Monoethanolamine, Corrosion Inhibitor, Aroma, Dye.

### Danger:

H315: Causes skin irritation.  
H318: Causes serious eye damage.  
H410: Very toxic to aquatic life with long lasting effects.  
P280: Wear protective gloves/protective clothing/eye protection/face protection.  
P301+P330+P331: IF SWALLOWED: Rinse mouth. Do NOT induce vomiting.  
P302+P352: IF ON SKIN: Wash with plenty of water.  
P305+P351+P338: IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.  
**Storage:**  
P403+P233: Store in a well-ventilated place. Keep container tightly closed.  
P235: Keep cool.  
P410: Protect from sunlight.  
**Contraindications:**  
Conditions to avoid: Heat.  
**Disposal:**  
Do not discharge concentrate into drains or rivers.

*Any serious incident related to this medical device must be reported to the manufacturer and the competent authorities.*

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