

Dual-Cure Core Build-Up, Endodontic Post Cement & Dentin Replacement Material



# CORE-FLO<sup>TM</sup> DC



# **Dual-Cured Flowable Core Build-Up/Dentin Replacement Material**



CORE-FLO DC is a versatile dual-cured, fluoride-containing core material that is ideal for core build-ups, post cementation and dentin replacement. CORE-FLO DC is radiopaque and available in two esthetic shades, NATURAL/A1 and OPAQUE WHITE to meet all of your direct and indirect restorative requirements.

- HIGH STRENGTH AND LOW WATER SORPTION
  ASSURES A STRONG FOUNDATION FOR CROWNS
  AND DIRECT RESTORATIVES CUTS JUST LIKE DENTIN
- LOW SHRINKAGE PROPERTIES ALLOW IT TO BE USED EFFECTIVELY AS A DENTIN REPLACEMENT MATERIAL
- **EXCELLENT FLOWABILITY** VOID-FREE FLOW MAKES IT IDEAL FOR POST CEMENTATION



 Failing restoration with recurrent decay and enamel fractures.



2. After isolation with dam and wedges, old restoration and decay were removed



 A sectional, wedge, and ring were placed before selectively etching the enamel. All-Bond Universal® was applied, air thinned, and light-cured, followed by Core-Elo DC.



4. The dual cure chemistry provides reliable polymerization while providing a high strength and dentin-like core.



5. The molar was prepped for a zirconia crown.



6. The zirconia crown provides an excellent restorative service for the patient.

Dentistry courtesy of Dr. Jack Griffin.

## **STACKABLE**





# CORE-FLO<sup>TM</sup>DC Lite




# **Self-Leveling Dual-Cured Core Material**



CORE-FLO DC Lite is a dual-cured, fluoride-containing core material that is dispensed with an auto-mix dual-syringe. CORE-FLO DC Lite's optimal self-leveling viscosity allows for excellent adaption resulting in gap-free margins when replacing natural dentition with a direct core build-up. In addition, the clinician can cement the post and build-up the core. CORE-FLO DC Lite is available in NATURAL/A1 and OPAQUE WHITE shades.

## **Dentin Shear Bond Strength**


The combination of CORE-FLO DC or CORE-FLO DC Lite with UNIVERSAL PRIMER™ offers unsurpassed bond strengths to dentin, enamel and fiber post.

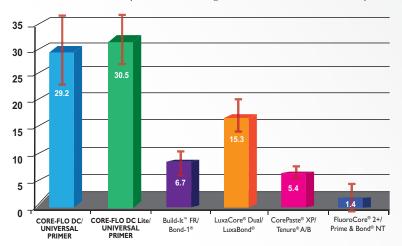

Core Material S/C. Adhesive placed per manufacturers instructions. Data on file.

Trademarks are property of their respective owners.

## **SELF-LEVELING**

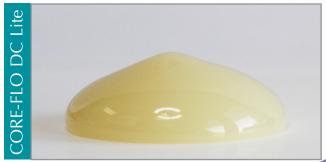



1. Pre-operative photo.



2. Amalgam and gross decay excavated



3. After adhesive is placed, Core-Flo DC Lite is placed into the preparation.



4. Final preparation.



5. Top view of final preparation.




6. Final post operative bitewing #19.

Dentistry courtesy of Dr. Adamo Notarantonio

# CORE-FLO DC Lite PHYSICAL PROPERTIES:

Working Time/Setting Time (S/C)	1 / 7 Minutes at Room Temp.
Compressive Strength	229 / 259 Mpa (LC / SC)
Flexural Strength	127 / 93 Mpa (LC / SC)
Radiopacity	1.8mm

### **POST CEMENTATION AND CORE BUILD-UP TECHNIQUE\***

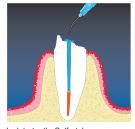


 Isolate tooth. Self-etch or total-etch post space. Rinse thoroughly.



**5.** Determine the post length according to the manufacturer's instructions.




 Continue to express CORE-FLO DC Lite around the post and buildup the core.

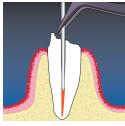

 Remove excess water with a brief burst of air. Remove any pooled water.




Coat the post with UNIVERSAL PRIMER, air dry.

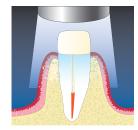

Light cure.




Dispense UNIVERSAL PRIMER. Mix adhesive.

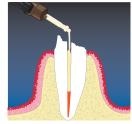


Inject CORE-FLO DC Lite into the canal.



**11**. Continue with final preparation for the impression.

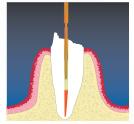

Scrub 2 coats of UNIVERSAL
PRIMER into the canal. Air dry to
remove excess solvent.

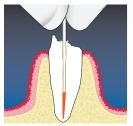

 Coat the apical end of the post with CORE-FLO DC Lite. Seat the post into the canal gently.

\* Refer to complete instructions for detailed techniques.

M-23412K

AB-17410K

## **ORDERING INFORMATION**

#### **CORE-FLO DC**

### CORE-FLO DC w/UNIVERSAL PRIMER

1 Dual-Syringe CORE-FLO DC Shade: NATURAL/A1 (8g), OPAQUE WHITE (8g),

1 Bottle ea. UNIVERSAL PRIMER Parts A&B (6ml ea.), 1 Dispensing Gun, Accessories, Instructions

CORE-FLO DC - 1 Dual-Syringe (8g), Accessories, Instructions

NATURAL/A1 A-23011P
OPAQUE WHITE A-23012P

### **CORE-FLO DC LITE**

### CORE-FLO DC LITE w/UNIVERSAL PRIMER Intro Kit

1 Dual-Syringe CORE-FLO DC LITE Shade: NATURAL/A1 (8g), OPAQUE WHITE (8g),

1 Bottle ea. UNIVERSAL PRIMER Parts A&B (6ml ea.), Accessories, Instructions

CORE-FLO DC LITE - 1 Dual-Syringe (8g), Accessories, Instructions

NATURAL/A1

OPAQUE WHITE

A-17801P

A-17803P

**TO ORDER CALL:** 

1-800-247-3368 • www.bisco.com



MC-2233CF Rx Only