# VOCO

## Amaris® / Amaris® Flow

Instructions for use  
MD EU Medical device

In accordance with DIN EN ISO 4049

### Product description:

**Amaris/Amaris Flow** is a highly aesthetic, light-curing, radiopaque restorative which contains 80 w/w % filler (individual mass shades HT and HO 64 w/w %) in a methacrylate matrix and cures under blue light. **Amaris** is available in screw syringes and Caps, **Amaris Flow** in syringes. Caps and syringes are available for direct applications. **Amaris/Amaris Flow** is used in combination with a dentine-enamel bond.

### Shades:

Base shades (opaque)

**O1, O2, O3, O4, O5, O Bleach**

provide the translucency and chroma of natural dentine.

Enamel shades (translucent)

**TL** (Translucent Light), **TN** (Translucent Neutral) and **TD** (Translucent Dark)

provide the translucency of natural enamel.

Special shades

**HT** (High Translucent) and **HO** (High Opaque) are flowables.

### Indications:

- Aesthetic restoration of class III, IV and V cavities in anterior teeth
- Aesthetic restoration of class I, II and V cavities in posterior teeth
- Reconstruction of traumatically damaged anterior teeth
- Direct and indirect composite veneers and facing of discoloured anterior teeth
- Shape and shade correction for improved aesthetics
- Repairs of highly aesthetic porcelain, e.g. veneers
- Splinting of anterior teeth

### Contraindications:

**Amaris/Amaris Flow** contains methacrylate and BHT. **Amaris/Amaris Flow** should therefore not be used in patients with a known hypersensitivity (allergy) to these ingredients.

### Patient target group:

**Amaris/Amaris Flow** is suitable for use in all patients without any age or gender restrictions.

### Performance features:

The product's performance features satisfy the requirements of the intended use and the relevant product standards.

### User:

**Amaris/Amaris Flow** should only be used by a professionally trained dental practitioner.

### Use:

**Preparation:**  
Clean the teeth being treated. If applicable, mark occlusal contact points. Bring material to room temperature before use. Minimal separation facilitates the shaping of the approximal contacts and placement of the matrix.

### Shade selection:

The components of **Amaris** are designed to provide first-class aesthetic restorations of large anterior defects using the layer technique. The palatal-vestibular layering technique is recommended in particular.

To select a shade moisten the tooth and the shade samples and then compare the enclosed shade guide to the cleaned tooth in natural light before administering anaesthesia.

Use the cervical area of the natural tooth or the shade of the moistened dentine, as visible in the cavity, as a guide for selecting the most suitable base shade (**O1-O5**). Refer to the incisal, occlusal or approximal area of the natural tooth to select the enamel shade (**TL/TN/TD**).

**Note:** Begin by applying a layer of the selected opaque shade to the defect (direction oral to vestibular) in cavity classes III, IV and V. The overall impression of the restoration in terms of shade can be influenced and adjusted through the selection of translucent shades for the finishing vestibular layer. The opaque base shade can be **darkened with TD, confirmed with TN and lightened with TL**.

In cavity classes I and II apply a layer of the corresponding base shade to dentine following the customary layer technique. Afterwards, apply a translucent shade as the enamel layer to finish the restoration.

With the assistance of the shade samples for the shades **HT** and **HO**, an impression of high transparency and high opacity for the individual shades can be achieved.

Using the shade **HT** (High Translucent), especially in the incisal area, allows for creating a perfectly individualised restoration.

The shade **HO** (High Opaque) can be used for masking surfaces that are discoloured, e.g. iatrogenic tooth discolouration, which is important for extensive defects.

### Cavity preparation:

In principle, the cavities should be prepared according to the rules for adhesive restorative treatment; preparation should be minimally invasive in order to protect healthy dental hard tissue. For anterior restorations, angle off the enamel margins. Then clean and dry the cavity. Caries-free lesions in the cervical area do not need to be prepared; thorough cleaning is sufficient in this case.

### Creating a dry working field:

Ensure that the work area is sufficiently dry. Use of a rubber dam is recommended.

### Pulp protection:

A suitable pulp-protective liner that can be covered with a stable cement, if necessary, should be applied in close proximity to the pulp.

### Bonding materials:

**Amaris/Amaris Flow** is applied with a dentine enamel bond, using the adhesive technique. Any light-curing bonding materials can be used. Observe the relevant instructions for use during preparation (etching technique) and processing.

### Application:

For cavities in approximal areas apply a matrix. Using a translucent matrix is beneficial (smooth surfaces).

Apply the desired shade of **Amaris** (see shade selection) and adapt it to the cavity with a suitable instrument. For fillings of more than 2 mm (shade HO: more than 1 mm), apply and polymerise in layers.

**Amaris Flow Syringe:** Lock the enclosed intraoral tip on the syringe with a clockwise turn and directly apply **Amaris Flow**.

The attached application tip is intended for single use only. Remove it after use and seal the syringe again tightly with its original cap. The use of a protective barrier is recommended to protect the syringe against contamination with bodily fluids, dirt from hands or oral tissue.

**Caps:** Insert Caps into the opening of the dispenser following the respective instructions for use. Turn Caps in the desired direction and remove the protective cover. Apply **Amaris** directly into the cavity by slowly and evenly pressing the levers of the handle together.

### Light-curing:

Conventional polymerisation devices are suitable for light-curing the material. Curing time if an LED/halogen lamp with a light output of at least 500 mW/cm<sup>2</sup> is used is

<b>Amaris</b> Base shades O1-O5, O Bleach	40 s
<b>Amaris</b> Enamel shades TL/TN/TD	10 s
<b>Amaris Flow</b> High translucent HT	10 s
<b>Amaris Flow</b> High opaque HO	40 s

Bring the light aperture as close as possible to the light-curing unit as otherwise the curing will be less effective. Inadequate curing can cause discolouration and complaints.

### Finishing:

The restoration can be finished and polished immediately after removing the matrices (e.g., fine or extra-fine diamond burs, polisher). The tooth should be fluoridated as a final step.

### Warnings, precautionary measures:

- No known side effects. However, sensitisation in hypersensitive patients cannot be ruled out.
- Phenolic substances, especially products containing eugenol and thymol, lead to curing failures in restorative composites. Consequently, the use of zinc oxide/eugenol cements or other materials containing eugenol should be avoided in combination with restorative composites.
- Each **Amaris** Caps unit should be used for one patient only and should be applied exerting uniform but not excessive pressure.
- Our information and/or advice do not relieve you of the obligation of checking that the products supplied by us are suitable for their intended purpose.

### Constituents (in descending order according to content):

**Amaris:** Barium aluminium borosilicate glass, prepolymer, UDMA, BisGMA, TEGDMA, methacrylated silica, fumed silica, initiators, stabilisers, pigments  
**Amaris Flow:** Barium aluminium borosilicate glass, TEGDMA, UDMA, BisGMA, fumed silica, initiators, stabilisers, pigments

### Storage instructions and application method:

Storage at 4°C-28°C. Seal the syringes immediately after removing the material in order to avoid exposure to light and subsequent polymerisation. Do not use after the expiry date.

### Disposal:

Dispose of the product in accordance with local regulations.

### Reporting obligation:

Serious events such as death, temporary or permanent serious deterioration of a patient's, user's or other person's health condition and a serious risk to public health that arise or could have arisen in association with the use of **Amaris/Amaris Flow** must be reported to VOCO GmbH and the responsible authority.

### Note:

The Summary of Safety and Clinical Performance of **Amaris/Amaris Flow** can be found in the European database on medical devices (EUDAMED - <https://ec.europa.eu/tools/eudamed>). Detailed information can also be found at [www.voco.dental](http://www.voco.dental).

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Schmelzfarben (Translucent)

**TL** (Translucent Light), **TN** (Translucent Neutral), **TD** (Translucent Dark)

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VC 60 BB1930 E1 1223 99 © by VOCO