



Seek and Find

Sable™ Seek® & Seek®
CARIES INDICATORS

Be certain you've identified all demineralized dentin. With highly visible colors and mess-free syringe delivery, Sable Seek and Seek caries indicators make finding caries a simple process of seek and find.




800.552.5512 ULTRADENT.COM


Sable™ Seek® & Seek®

CARIES INDICATORS


- Highly visible Sable Seek and Seek caries indicators clearly distinguish demineralized dentin for efficient, confidence-inspiring removal
- Precise, mess-free syringe delivery
- Patient-friendly dyes
- The unique green color of Sable Seek indicator prevents overexcavating deep caries, which can lead to pulp exposure
- The vibrant red color of Seek indicator easily identifies root canal orifices

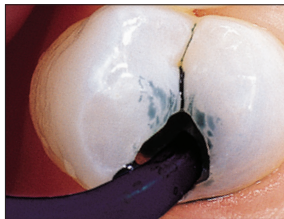


1. Apply Sable Seek indicator with Black Mini Brush tip.

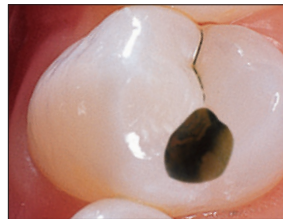

2. Rinse with air/water and suction. Carious dentin is easily identified.

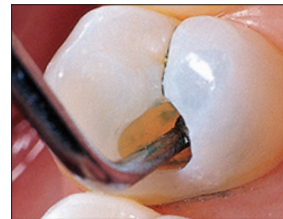

3. Remove green-black color (carious dentin) with slow-speed round bur or excavator. To control overexcavating near the pulp, remove final portion of caries with hand excavator.

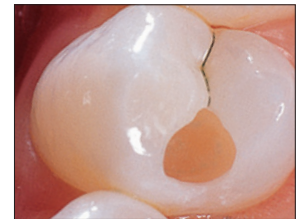

4. Reapply. Rinse and verify complete caries removal.

209 **Seek Kit**
4 x 1.2 ml syringes
20 x Black Mini Brush tips



210 **Seek Refill**
4 x 1.2 ml syringes

1804 **Seek Econo Refill**
20 x 1.2 ml syringes

233 **Sable Seek Kit**
4 x 1.2 ml syringes
20 x Black Mini Brush tips



234 **Sable Seek Refill**
4 x 1.2 ml syringes

1805 **Sable Seek Econo Refill**
20 x 1.2 ml syringes

SHELF LIFE: 36 MONTHS.

SHELF LIFE: 36 MONTHS.



800.552.5512 ULTRADENT.COM

 © 2019 Ultradent Products, Inc. All rights reserved.
68479.8 101118

 **ULTRADENT**
PRODUCTS, INC.

Improving Oral Health Globally