Geistlich
Biomaterials

Manufacturer
Geistlich Pharma AG
Bahnhofstrasse 40
6110 Wolhusen
Switzerland
info@geistlich.com
www.geistlich-pharma.com

Geistlich
Bio-Oss[®]

Geistlich
Bio-Oss Pen[®]

937111/1111

LEADING REGENERATION

Content

English	2	Bahasa Melayu	110
Deutsch	8	Malti	116
Français	14	Nederlands	122
Italiano	20	Norsk	128
Español	26	Polski	134
Български	32	Português	140
Česky	38	Română	146
中文 (中华人民共和国)	44	Русский	152
Dansk	50	Slovenčina	160
Eesti	56	Slovenščina	166
ελληνική	62	Srpski	172
עברית	68	Suomi	178
Hrvatski	74	Svenska	184
Íslenska	80	ภาษาไทย	190
한국어	86	Türkçe	198
Latviešu	92	Українською мовою	204
Lietuvių	98	Distributors	212
Magyar	104		

English

Geistlich Bio-Oss®

Geistlich Bio-Oss Pen®

Natural bone mineral for bone grafting

Composition

Geistlich Bio-Oss® is a natural bone mineral of bovine origin. Geistlich Bio-Oss® is available as granules of spongy bone and as a spongy-bone block. The granules are also available in an applicator as Geistlich Bio-Oss Pen®. The highly purified osteoconductive mineral structure is produced from natural bone in a multi-stage purification process, adhering to the strictest safety regulations. Because of its natural origin, Geistlich Bio-Oss® is chemically as well as structurally comparable to the mineralised human bone. Sterilisation of Geistlich Bio-Oss® is carried out by γ -irradiation.

STERILE	R
---------	---

Properties / Effects

The inorganic bone matrix of Geistlich Bio-Oss® has a macro- and microporous structure similar to human spongy bone. Due to the large interconnecting pore volume and the natural composition, the formation and ingrowth of new bone at the implantation site is encouraged. In the course of time Geistlich Bio-Oss® is partially remodeled by osteoclasts and osteoblasts (physiological remodeling). Due to its properties, Geistlich Bio-Oss® is a viable alternative to autologous bone in suitable defects.

Indications

Geistlich Bio-Oss® is recommended for filling of bone defects and bone augmentation, e.g.

- > Augmentation/reconstruction of alveolar ridges
- > Filling of extraction sockets
- > Implantology: preparation of implant sites, filling of bone dehiscences and sinus floor augmentations
- > Periodontology: filling of bone defects, support of the membrane during guided tissue regeneration (GTR)

Geistlich Bio-Oss® block is especially recommended for filling of large bone defects.

Instructions for use

The general principles of sterile handling and patient medication must be followed when using Geistlich Bio-Oss®.

- > Complete elimination of granulation tissue following exposure of the defect.

> Application:

a) Geistlich Bio-Oss Pen®: please see pictograms on the package label or on the last page of the booklet to follow the instruction below.

1. Holding the Geistlich Bio-Oss Pen® firmly, tear off the protective seal (A) and unscrew the protection cap (B). This exposes the applicator and filter cap.
2. Do not remove the red/green filter cap from the applicator at this time.
3. For complete wetting of Geistlich Bio-Oss®, hold the applicator with both hands and place the filter cap into the sterile physiological saline solution or the patient blood. Completely retract the plunger. For 0.5 cc applicator, one retraction is sufficient to complete wetting of Geistlich Bio-Oss®. For 1.0 cc and 1.5 cc applicators, depress the plunger, using thumb pressure, to eject the wetting fluid without

compressing Geistlich Bio-Oss®. Re-wet with the same wetting fluid by retracting and depressing the plunger approximately 5 times. Alternatively, use a separate syringe (not included) to wet Geistlich Bio-Oss®. Directly inject the wetting fluid of choice through the filter cap into the Geistlich Bio-Oss® granules.

4. Once wetted, gently depress the plunger to eject any excess fluid.
5. Remove the filter cap from the applicator (A). Attach the curved applicator tip by screwing it on to the end of the applicator (B). The applicator is now ready for use. Using thumb pressure, depress the plunger and apply Geistlich Bio-Oss® directly to the surgical site.

b) Geistlich Bio-Oss® granules in vials: mixing of Geistlich Bio-Oss® granules with the patient's blood or with physiological saline solution before the implantation.

Granules are placed into the defect, using sterile instruments (spatula or spoon).

- c) Geistlich Bio-Oss® block: the block may be adapted to the desired size with a scalpel and placed into the bone cavity. If necessary, the block is moistened with the patient's blood or with physiological saline solution.
- > In situ modeling may be performed with a sterile spatula or other suitable instrument.
 - > It is advisable to cover Geistlich Bio-Oss® with a membrane barrier (e.g. Geistlich Bio-Gide®).
 - > When closing the wound, the soft tissue flap should completely cover the implanted Geistlich Bio-Oss®, and should be fixed by sutures.
 - > If primary wound closure cannot be achieved completely, further mobilisation of the flap (incision through the periosteum) should be performed or

wound covering should be achieved with a membrane (e.g. Geistlich Bio-Gide®).

Special instructions for use in periodontology

- > A basic requirement for successful periodontal treatment includes control of any bacterial infection as well as thorough oral hygiene. It is therefore advised that, preceding the surgical intervention, there be a hygiene phase, which would include proper instruction for the patient. A postoperative maintenance phase can ensure long-term therapeutic success.
- > Besides plaque control, the filling of periodontal defects with Geistlich Bio-Oss® requires successful local treatment of the periodontal lesion (root planing, debridement), prior to the implantation. The defect should be covered with a membrane (e.g. Geistlich Bio-Gide®) for optimal tissue regeneration.

Limitation of use / Precautions

Contraindications

Geistlich Bio-Oss® should not be used in the presence of infected wounds.

Precautions

Geistlich Bio-Oss® should only be used by trained dentists and surgeons.

Geistlich Bio-Oss® should be used with special caution in patients with

- > acute or chronic infection (e.g. osteomyelitis) at the surgical site;
- > uncontrolled metabolic disorders e.g. diabetes, osteomalacia, thyroid disorder, severe renal or hepatic disorder
- > prolonged corticosteroid therapy
- > autoimmune disease
- > radiotherapy
- > heavy smoking

In order to ensure the regeneration of bone, Geistlich Bio-Oss® should only be implanted in vital bony tissue and in direct contact with the host bone (if necessary with microfrac-

ture of the bone surface). For larger defects, the addition of autologous spongy bone or bone marrow may improve the regeneration process. From experience, mobility by mechanical loading (compression loading) or insertion of implants (two-stage procedure) in the augmented area should be avoided until several weeks after the insertion of Geistlich Bio-Oss®. From experience, mechanical loading (compression loading) of Geistlich Bio-Oss® augmented areas is possible after 6 months at the earliest. According to this, the appropriate timing of implant insertion depends on the residual local bone volume.

Side effects

Incompatibility reactions with Geistlich Bio-Oss® cannot be totally excluded. Possible complications which may occur with any surgery include swelling at the surgical site, flap sloughing, bleeding, local inflammation, bone loss, infection, or pain.

Pregnancy / Lactation

Data on the use of the product during pregnancy or lactation are not available. For safety reasons: women who are pregnant or breastfeeding should not be treated with Geistlich Bio-Oss®. The safety and efficacy of Geistlich Bio-Oss® have not been investigated in children before skeletal maturity.

Other Information

The content of the sterile package is intended for single use.

Re-sterilisation is not possible. Discard any unused material. Sterility and function are no longer guaranteed in the event of reuse.

Do not use the product after the indicated expiry date.

Store in a dry place and at room temperature (15–25° C).

The product is sterile, if the packaging is unopened and undamaged. If the sterile packaging has been damaged, the product must not be used.

 **Manufacturer**
Geistlich Pharma AG
Bahnhofstrasse 40
6110 Wolhusen
Switzerland

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